

LEVELING PLATE

DESCRIPTION: Designed by Polymedco, this Leveling Plate ensures That the *Sediplast*® rack is level and that accurate ESR results will be obtained. Comprised of solid aluminum, the Leveling Plate is fitted with a mounted circular level and adjustable legs. Each 11" x 9", is polished and anodize treated to provide years of laboratory use.

INSTRUCTIONS FOR USE:

Place rack containing *Sediplast*® pipettes (tubes) and vials onto the plate. Or place the Sedimat on the plate.

Level plate by turning the black adjustable legs so that the air bubble moves into the small centering circle on the level.

When the bubble is located exactly within the circle, the plate is level.

 **POLYMEDCO, Inc.**

510 Furnace Dock Road, Cortlandt Manor, NY 10567 800-431-2123 FAX: 914-739-5890

PN51515-00
REV 04/26/05