

Patient Identifier Tags for Sediplast® Tubes Instructions for Use

Step 1. Hold Patient Identifier Tag with the cap holder facing downwards

Step 2. Starting at the bottom of the Sediplast tube, slide the Patient Identifier Tag up until it reaches the cap of the Sediplast tube

Step 3. Push the Patient Identifier Tag onto the bottom of the Sediplast tube cap until it snugly snaps into place

Step 4. Apply handwritten and/or patient ID adhesive label to the top of the Patient Identifier Tag